

Cambivo Patella Knee Strap Instruction

About Patella Knee Strap

The knee straps are often referred to as Jumpers Knee Straps or Patella Straps and they are simple but effective. Knee bands are designed to add pressure through the patella tendon and support the kneecap, therefore improving patella tracking (knee cap movement).

Use for:

Osgood–Schlatter disease (OSD), patella dislocation, patellofemoral pain syndrome, Arthritis, chondromalacia, jumper' s knee, patella tendonitis, runner's knee

Suit for

Activities that cause a lot of pressure on the knees like football, basketball, hiking, running, skipping, Zumba, up and down stairs, etc.
You can also treat it as a rehabilitation product after a knee injury. Support your knees and reduce the risk of re-injury.

Functions

- Normalize joint range of motion
- Provides instant strength and stability
- Improve patellofemoral (knee cap) alignment
- Minimize your chance of re-injury
- Reduce the wear of the knee joint
- Release stress for your knees
- Reduce pain and inflammation

Package Contents

2 x knee straps with long straps suitable for adults
2 x bonus short straps which are suitable for kids

Size Chart

Measure the circumference at 1 inch(2.5cm) below your patella and refer to below size chart.

Size Chart	
Knee Circumference	Size
28cm-45cm(11-18")	Regular
45cm+ (18"+)	Plus

·Regular size is suitable for most people.

How to Wear It

Straighten your legs and replax your thighs



Step 1: Place the knee strap about 1 inch(2.5cm) below the knee cap.



Step 2: Affix one end of the strap to the back. Tighten with the other end until the strap is snug and comfortable.



Step 3: Affix the other end of the strap. The entire surface of the brace is grippy and you may overlap the strap if needed.

Care Instructions

- Hand wash in mild detergent cold or warm water
- Rinse thoroughly and drip dry by air
- Do not bleach
- Do not put in the washer or dryer
- Do not dry clean
- Do not iron

Composition Material

60%SBR (neoprene), 30% polyester fiber, 10% spandex

DISCLAIMER: This product is not intended to diagnose, treat, or cure any disease. If you have unfortunately suffered a meniscal tear, the patella strap cannot cure it. It is recommended that you choose a more suitable protective gear under the guidance of your doctor.

ALLERGY NOTICE: These products may contain natural rubber latex and may cause an allergic reaction. If an allergic condition occurs, stop using it immediately. If the allergic is intensified or the pain persists, please consult your doctor.

Warranty Information

Terms & Policy

Cambivo warrants all products to be of the high quality in material, craftsmanship and service for a minimum of 1 year, effective from the date of purchase. Replacement or refund is available for any product found manufacturer flaws and sizing problem.

This warranty does not apply to the following:

- Damage due to abuse, accident, alteration, misuse, tampering or vandalism.
- Improper or inadequate maintenance.

You can register to extend your warranty for additional 1 year!

www.cambivo.com/warranty

Customer Support

If you encounter any issues or have any questions regarding your new product, feel free to contact our Customer Support Team. We're here for you.

 support@cambivo.com

*Please have your order number ready before contacting customer support.

Cambivo Best Sellers



Visit Amazon store to check all of our products.

Be part of our Community!

- Receive FREE Tips, Product Information, and eBooks.
- Win FREE products and exclusive deals.



@CambivoInc



@Cambivo



Win FREE Products



Exclusive Deals